LING CC-19

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The "LING CC-19" blocking bandage is a revolutionary reality that facilitates, from any point of view, the application of blocking bandages for the professional and the daily use of the patient to whom it is applied.

The product consists mainly of 2 elements:

- **BASE ELEMENT OR SUPPORT**

It is which is a fiberglass fabric of recycled origin, with a taffeta ligament imitating traditional bandage, with non-flammable edges. With very high resistance and a specific finish to react on element B and together with the activator, which is when it comes into contact with water.

- **HARDENER ELEMENT**

corresponds to the product that impregnates the base element and is supplied already applied to it, facilitating the tasks of preparation for application to the patient. This impregnation of the base is a compound of recycled polyurethane resins and resins and plant origin resins (without cutting down trees for its production)

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The functionality of the compound corresponds to the union of the 2 components with the reaction of the water, creating a product with characteristics of hardening and resistance features, nowadays unrivalled.

It has been a very long way to achieve a different product from what currently exists, with all the advantages described below, which have been laboratory tested and directly on patients, with really surprising and satisfactory results.

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✓ **RESISTANCE:** Tear and deformation resistance tests have been performed. Deformation (by a percentage of 10%) is obtained with a load at 120 kg. and with a central pressure on a 10 x 10 cm base applied in the center of the immobilization bandage. Breakage is achieved with a pressure of 185 kg. Equally, centralized in a 10 x 10 cm. base applied in the center of the immobilization bandage.

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✓ **FLEXIBILITY:** In a habitual immobilization, no flexibility is appreciated that can cause any inconvenience in the result of the blockage of a limb. In the case of a pressure forced by the pressure of a person, the variation by flexibility is negligible, not exceeding 0.02 %

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✓ **WEIGHT:** The weight reduction compared to a conventional blocking bandage (plaster) is of 95% i.e. a conventional plaster immobilization system that can range between 2000 / 2200 gr. With this system has a weight of 105 gr.

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✓ **RADIOLOGY:** The use of our immobilization system does not interfere with the radiological tests for the observation of the affected parts.

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✓ **CUSTOMIZATION OF COLORS:** During the spinning process, it is possible to customize the bandages with the corporate colors of each client, so that it can be woven directly to color. All the colours used in the dyeing of the yarns are free of contaminating elements.

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✓ **RECYCLING OF USED BANDAGES:** We can carry out the selective collection of solid waste corresponding to bandages for shredding and subsequent use in reinforcement components for the construction industry and reinforcement of asphalt roads.

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✓ **EXPIRY DATE:** Each unit has the expiration date printed on the packaging, which is 3 years from the date of manufacture.

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1 Apply 1 or 2 layers of "LING CT-19" cotton tubular (Cotton tube) on the affected part, using additional filler at the discretion of the specialist.

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3 Before opening the container, choose the right size. Opening the package by mistake and its contact with the humidity of the environment for approximately 10 minutes causes the immobilization bandage to solidify partially, making it unusable.

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4 Immerse the immobilization bandage in water for 4-6 seconds and squeeze 2-3 times. The warmer the water, the shorter the setting time.

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5 Wrap the spiral immobilization bandage, overlapping half or two-thirds of the width of the subsequent layer bandage onto the anterior layer. Be careful to maintain proper tension and avoid wrapping too tightly. Resistance can be determined by the number of layers wrapped. It is only necessary to wrap 3 layers in the non-weight bearing position and 5 layers in the weight bearing position.

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6

If more rolls of moulds are used at the same time, make sure there is a good bond between the bandages, this bonding is done by using water and rubbing in the bond/superposition zone.

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7

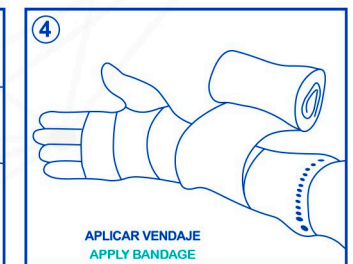
Smooth and scrub the surface to achieve good adhesion between coats. The immobilization bandage begins to solidify within 3-5 minutes after immersion in water. Therefore, the operation should be completed in 3-5 minutes. Patients should not move the affected part before 10 minutes.

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8

Before opening the container, choose the right size. Opening the package by mistake and its contact with the humidity of the environment for approximately 10 minutes causes the immobilization bandage to solidify partially, making it unusable.

RESUMEN GRÁFICO INSTRUCCIONES DE USO GRAPHICAL OVERVIEW OF THE INSTRUCTIONS FOR USE

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Avoid knocking and damaging the container, as this will cause the bandage or immobilization splint to solidify prematurely in the same container. If it has partially solidified, it can no longer be used.

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The affected part must first be wrapped with a gauze bandage or with the cotton tubular "LING CT-19".

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Surgical gloves should be worn while using the cast to prevent polyurethane resin from adhering to unprotected skin and clothing. If it adheres to the skin, it should be removed immediately. If it has solidified, wait for it to fall off on its own.

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Any type of bandages or immobilization splints should be applied by professionals in traumatological medicine, since improper or erroneous use can create orthopedic problems and malformations in the bone system.

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Open only one container at a time and use it immediately, as humidity can solidify the bandage and render it unfit.

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The solidification time is directly related to the temperature of the water. The hotter the immersion and subsequent rubbing water, the shorter the solidification and curing time.

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The most relevant characteristics coincide with the product "LING CC-19" both in its manufacture and in its application. The only difference is in the coating of the fiberglass, as it is coated with a layer of non-woven punched, to avoid (if desired) direct contact of the splint with the skin.

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It is a totally recycled textile product of 100% cotton composition, for use as an intermediate element between the application of splints or immobilization bandages and the patient's skin.

It is made with ribbed knit ligament, which offers maximum comfort, as it does not offer any pressure on the area where it is applied.

Indesmallable, which does not produce uncomfortable threads for patient use.

It is supplied in bulk rolls of natural cotton colour without any type of dyeing. The piece is cut as needed, without producing unnecessary waste.

SUMINISTROS MÉDICOS DE CALIDAD
QUALITY MEDICAL SUPPLIES